Reusable Cloth Mask
Pattern-Hack Instructions by, Kaili Reynolds
Pattern pieces from www.avera.org
Translated to Spanish by Joe Schirmer
All seams are $\frac{1}{4}$ inch. Seam allowances are already made.

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
Step 1: Download and print pattern pieces face 1 & face 2 only

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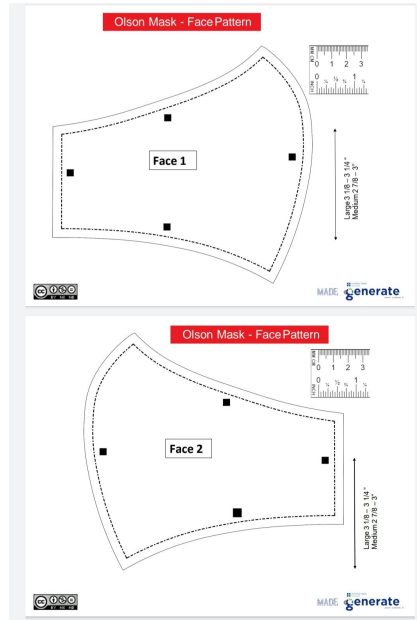<https://www.avera.org/app/files/public/76443/Olson-Mask-with-Pattern.pdf>

Step 2: Trace pattern onto **folded** cotton fabric (quilting or t-shirt fabric work best) and cut. You should have 4 pieces after cutting.

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Step 3: Pin face piece 1 & face piece 2 right sides together and sew along curved edge. Top stitch only. Repeat step for the other two pieces.

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Step 4: Pin the two large face pieces you just made, right sides together and sew along all edges **except one side.**

Leave one side open and then turn mask right side out when you are finished.

Do not sew all the way to any edge, make sure to leave $\frac{1}{4}$ inch from seam to edge of fabric. Cut extra fabric off corners once sewn.

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Sew all edge except
this side!!

Step 5: After turning mask right side out, tuck in unsewn edge $\frac{1}{4}$ inch. Pull edges of mask through one hair band each. These will be worn on the ears. Pin edges down and sew. Make sure to back stitch & lock stitch as there will be a lot of tension when worn.

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Step 6: Wash or iron to prevent spreading germs to the owner of the mask. Instruct the owner to do the same before wearing. Good luck and be well!

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